

Biomedical Systems

Aerospray® TB Slide Stainer/Cytocentrifuge

Series 2



SEE WHAT YOU NEED TO SEE

The only slide stainer with built-in cytocentrifugation and programmable staining capabilities.




AEROSPRAY® TECHNOLOGY GIVES SUPERIOR STAINING RESULTS

- Superior staining results in 4-6 minutes with an easy-to-use process
- Consistent, reliable, and standardized staining results across multiple labs
- Reduced cross-contamination with a fresh application of stain applied to every slide
- Minimized stain waste with an 18-month reagent shelf-life




High-Quality Slides for Superior Smearing

- Optimized for Aerospray Slide Stainers, Cytocentrifuges, and CellaVision® Digital Cell Morphology Analyzers
- High-quality white soda-lime glass enables smooth smears and even monolayers
- Blank or 45° clipped corner microscope slides

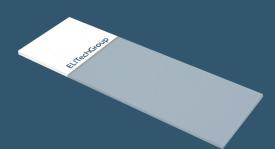

PROGRAMMING CAPABILITIES DESIGNED FOR PREFERENCES - SEE WHAT YOU NEED TO SEE

SELECT YOUR DEFAULT STAINING PROGRAM

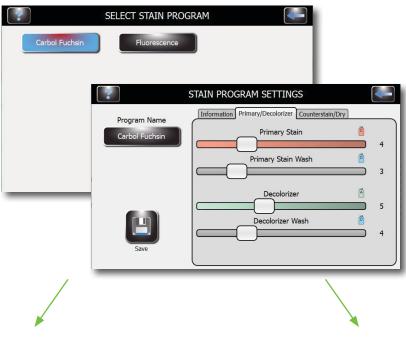

Carbol Fuchsin

Ziehl-Neelsen (ELITe ZN technology), without the dangerous fumes linked to the traditional heating phase.


Fluorescence




Methylene Blue Counter Stain



Brilliant Green Counter Stain

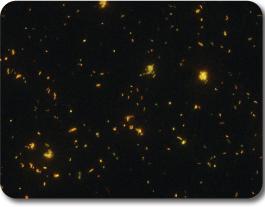

Auramine- Rhodamine Primary Fluorescent Stain

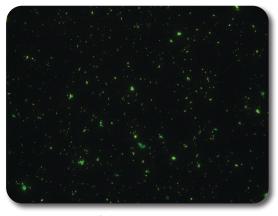

Auramine O Primary Fluorescent Stain



"Delivering innovative diagnostic solutions that enable better medical decisions."

Aerospray® TB Model 7722

•	
Slide Carousel Capacity	1 to 12 or 1 to 30 Slides, depending on carousel (carousel choice optional)
Cytocentrifuge Rotor Speed	100-2000 rpm (± 5%), user programmable
Dimensions (W x H x D)	57 cm x 25 cm x 54 cm (22" x 10" x 21")
Clearance Height (lid open)	58 cm (23")
Weight	17.5 kg (38.6 lb)
Electrical Requirements	100 to 240 VAC (±10%) @ 50-60 Hz, 200 watts maximum
Drain Connection	Outlet on rear panel
Safety	Lid interlock: lid must be closed for operation, and is locked down during carousel rotation.
Reagent Level Detect	Level detection on all reagent lines
Waste Level Detect	Level detection in waste bottle (optional)

Cytopro® Cytocentrifuge Rotor Model AC-160

Capacity	8 chambers and slides		
Weight	1.1 kg (2.5 lb)		
Dimensions (Dia. x H)	22.6 cm x 6.2 cm (8.9" x 2.4")		

	Catalog #	Description		Catalog #	Description
	MODEL 7722	AEROSPRAY® TB Slide Stainer/Cytocentrifuge	Ж	SS-061AF	TB Reagent A, Fluorescence Decolorizer, 500mL
	AC-160	Cytopro® Rotor	Ž	SS-061BP	TB Reagent B, Potassium Permanganate, 500mL
	SS-029	Aerospray® Nozzle Cleaning Solution, 355mL	FLUORES	SS-061CA	TB Reagent C, Auramine O, 500mL
	SS-029C	Aerospray® Nozzle Cleaning Solution Concentrate, 250mL		SS-061CAR	TB Reagent C, Auramine/Rhodamine (mod), 500mL
	SS-061M	TB Reagent P, Mycohold™ Cell Adhesive, 500mL		SS-161AF	TB Reagent A, Fluorescence Decolorizer Concentrate, 30mL
	SS-161M	TB Reagent P, Mycohold m Cell Adhesive Concentrate, 100 mL		SS-161BP	TB Reagent B, Potassium Permanganate Concentrate, 200mL
	SS-230	Aerospray® Stain Residue Solvent, 500mL			
	SS-249	Acid Fast Stain Control Slides, Pack of 50		SS-061A	TB Reagent A, Carbol Fuchsin Decolorizer, 500mL
	SS-266	Aerospray® PM Cleaning Solution, 500mL	点	SS-061BBG	TB Reagent B, Brilliant Green, 500mL
	SS-MeOH	Aerospray® Reagent Grade Methanol, 500mL	畄	SS-061BMB	TB Reagent B, Methylene Blue, 500mL
	AC-187	Barcode Scanner and Waste Level Detect Bundle	뷛	SS-061CEZ	TB Reagent C, ELITe ZN, 500mL
	AC-188	12-Slide Carousel, Aerospray® Stainers	ZIEHL	SS-161A	TB Reagent A, Carbol Fuchsin Decolorizer Concentrate, 30mL
	AC-189	30-Slide Carousel, Aerospray® Stainers		SS-161BBG	TB Reagent B, Brilliant Green Concentrate, 130mL
	SS-280	Clipped Corner Glass Microscope Slides, Frosted		SS-161BMB	TB Reagent B, Methylene Blue Concentrate, 130mL
	SS-287	Glass Microscope Slides, Frosted			



WORLDWIDE OFFICES

Please contact your sales representative for terms, conditions and product availability in your country.



Belgium & Luxembourg Brazil France Italy New Zealand

+33 1 41 45 07 10 T: +61 1800 815 098

T: +32 9 282 05 31 T: +55 27 3025 1415 T: +33 4 83 36 10 82 T: +39 02 48 40 35 42 T: +64 800 555 611

Germany Canada Serbia The Netherlands

T: +49 30 7809 588-0 T: +1 888 662 4925 T: +381 11 2467119 T: +31 313 430 500 T: +44 1442 869320 United States T: +1 435 752 6011







AEROSPRAYSTAINING.com aerospray@elitechgroup.com